



Instructions treatment chair PODO



Martini Beauty B.V.
Tel: 085-0450056
E-mail: info@martinibeauty.nl
www.martinibeauty.nl

(this manual is a copy of the original Dutch version)

Index

Carefully read the attached instructions and keep them for service after bought.

SAFETY REQUIREMENTS.....	3
INITIAL STARTING	3
APPLYING.....	4
OPERATE	4
CLEANING AND MAINTENANCE.....	5
TECHNICAL FAILURES AND SOLUTIONS.....	6
REPARATION AND MAINTENANCE.....	6
TECHNICAL DATA.....	7
CONDITIONS FOR TRANSPORTATION, STORAGE AND USE.....	7
DISCHARGING	7
CONTENT OF DELIVERY	7

Safety requirements

- Carefully read the instructions of the treatment chair before putting it into use. The instruction can contain important information what has to do with commissioning, handling and the maintenance of this device.
- By figuratively or different use then stated in the instructions, will all assurance and liability of the factory expire.
- Only the manufacturer or the manufacturers authorised expert can open the control section.
- When arrived always check if the treatment chair has no damage and/or has all the pieces. When in doubt please contact the customer service within 15 days.
- Use an undamaged power cable when connecting the chair. The cable must never be pinched off or bent and must never be in contact hot devices (e.g. heater or lamps).
- If you connect the chair with a damaged power cable, turn the device off, do not touch the cable, pull the plug out immediately. Contact the customer service. Never change the power cable yourself.
- Before use check if the electric current (V) is the same as stated.
- Never pull the cable out of its socket or take the power cable out with wet hands.
- Do not use this chair in humid places or transfer liquids.
- In case liquids or strange goods enter the control section by accident, directly turn the power off and pull the power cable out of its socket. Before using the device again contact our customer service.
- Always pull out the power cable before cleaning the device.
- If you want the chair to mount on the floor we can deliver the necessary tools.

Initial starting

1. Lifting the treatment chair in place:

- Always lift the chair with two persons.
Place one hand sideways at the front of the seat.
Place the other hand around the metal of the armrest.



1.1 Putting the treatment chair in place:

- Place the treatment chair on a smooth, stable foundation.
- With help of the screws(10) in the back of the footrest, the chair can be adjusted so it will be able to stand stable and straight.
- Remove the transport protection and other packaging materials, after the placement of the chair.

2. Connecting the chair to the electrical power network.

- Always check if the electricity (v) if the same as on the sticker.
- Put the power cable into the socket.

The treatment chair is now ready for use.

Applying

The Podo treatment chair is fully automatic.


The sitting height, tilt function and backrest work via three high performance motors. The settings are working via a multifunctional hand and foot control.

Because of the foldable arm rests and the low entry of ca. 27 cm, one can pleasantly take place on the chair or go off. The Podo treatment chair can be used for all Pedicure treatments.

Operate



1	Back and headrest
2	Armrest
3	Leg rest/feet rest
4	Lever to adjust leg rest
5	Handcontrol (standard)
6	Adjusting back side (up/down)
7	Tilt function (seat and back at the same time)
8	Up and down
9	Rotable (option)
10	Adjusting screws

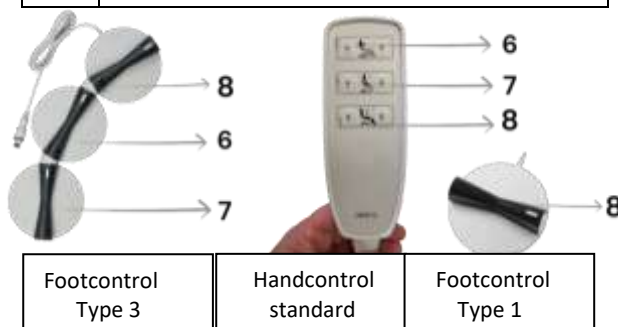

Control



CAUTION! Oppression danger! When adjusting the chair, the patient's arms must lean on the arm supports.

The feet control must not be placed underneath the sitting when adjusted.

CAREFUL! Injury danger! If the chair is at its highest massage position, balance the weight. The chair can drop forward if the chair is not balanced.

1. Adjust backside (1):

You can adjust the angle of the backside as follows:

- Press button ↑ (6) of the control, to push the back of the chair forward.
- Press button ↓ (6) of the control, to configure the back.

2. Adjusting the angle of inclination (7):

The angle of inclination will be adjusted,

- Press button ↓ (7) of the control, to bend the chair forward.
- Press button ↑ (7) of the control, to push the chair backwards.

3. adjusting the sitting height (8):

- Press button ↑ (8) of the control, to higher the treatment chair.
- Press button ↓ (8) of the control, to lower the treatment chair.

The chair can be adjusted higher or lower till the desirable height.



4. Adjusting leg en feet rests (3) :

The leg- and feet rests will be autonomously and manually adjusted from the sit- and back part.



Watch out! The feet rest must always be retracted by boarding and getting off. Even if the chair is on its lowest stand.

Turn outward:

- De leg rests can bend 90° to their sides.
- Push the leg rest manually to the inside or outside till a desirable position.

Extend leg section:

- Loose the outer lock screw of the leg rest, to move the leg rest in and out till the desired position.
- To fasten the leg rest, tighten the screw.

Leg section height adjustment:

- The leg rest locks itself when put on the desirable height.
- Use lever (4) underneath the leg rest to unlock the leg rest to lower it.
- Loose the outer lock screw of the feet rest, to move the feet rest in and out till the desirable position.
- To fasten the feet rest, tighten the screw

5. Adjusting arm rest:

Both armrests can fold 180° behind themselves.

- Pull the arm rests up.
- Pull the arm rests up.

Pulling armrests up:

- Pull the armrest up.
- Push the arm rest down until they click.

6. Turning off the treatment chair:

- Remove the plug out of its socket.

Cleaning and maintenance



Careful!

Possibility on getting an electric shock!

Always pull out the power cable out if it's socket before cleaning!

- The treatment chairs upholstery can be cleaned with warm water and a clean cloth. We recommend using microfiber clothes of cleaning gloves.
- Remove substantial polluted spots with mild cleaning products.
- Remove doggedly spots with disinfection- and cleaning foam.
- To prevent the infiltration of stains in the upholstery, we recommend to immediately remove them.
- Clean the frame of the treatment chair and the feet pedal with a light moist cloth. Be careful that no damp enters the control of the pedal.



Caution!

- When disinfecting interior, we recommend disinfect- and cleaning foam.
- Never rub disinfection products onto the upholstery.
- Do not use detergents as chlorine, carborundum, synthetic cleaning products, polishing wax, air freshener or aggressive cleaning products. These can damage the upholstery.
- Do not use products that contain oil or fat to clean the upholstery.
- Once a year the chair has to be checked for technical faults or cable breaks.

Technical failures and solutions

Failure	Cause	Solution
Automatically adjusting the treatment chair doesn't work.	Power cable is not connected right.	Check if the cable is not damaged. Be sure the cable is connected right.

By other failures than described. Contact us via our technical service. Tel: 085-0450056

Reparation and maintenance

The treatment chair normally does not need extra maintenance. In case problems appear, contact our customer service. Reparation and maintenance must only be done by the manufactory or our technical service. If a third party will change or repair, all assurances of the manufacture expire, and the safety will not be assured.

Technical data

Article number:	10001	
Spanning/AC:	100 V~/ 50-60 Hz	
Capacity:	200 VA	
Measurements	Width incl. the arm rests	69 cm
	Length	182 cm
	Length leg rest	55 cm
	feet section extensible	20 cm
	Sitting height	51-100 cm
	Depth of the seat	48 cm
Maximum carrying weight	175 kg	
Weight treatment chair	75 kg	

Conditions for transportation, storage and use

Only transport in the factory's original packaging.

Transportation- and storage temperature: -20 °C to +50 °C

Service temperature: +10 °C to +40 °C

Relative atmospheric humidity: Max. 90% not condense.

Prevent hefty temperature fluctuations in connection with condensation. Use the device after it has acclimated.

Discharging

Preserve the original package before sending out. Despatch unnecessary packaging, transport chair and belongings by designated authorities, following the former environmental rules.



All electro- and electronic devices must be separated from domestic refuse.

When noticing this symbol; a container with a cross through it, on a product, this falls under European guidelines 2002/96/EC and must not be provided together with domestic waste. These products must be brought to a waste collection facility. By proper despatching and separating of gathered used devices prevent environmental and health damage. It is a condition for reuse and recycling of defunct electricity and electronic devices.

Content of delivery

1 x including wiring and operating mechanism

EC DECLARATION OF CONFORMITY OF MACHINES

(according to Annex IIA of the Machinery Directive 2006/42 / EC)



We:

Martini Beauty B.V.

Stavangerweg 8A

9623 JC Groningen

The Netherlands

info@martinibeauty.nl

Tel. +31 85 0450056

Declare, under our sole responsibility, that the following installation:

1. Podo treatment chair
2. Podo / Beauty treatment chair
3. Beauty treatment chair

Be in accordance with the relevant parts of the following Directives and relevant parts of the following harmonized standards:

2006/42/EG – Machinery Directive – by applying relevant parts from the following standards:

NEN-EN-ISO 12100:2010

Safety of machinery – General principles of design – Risk assessment and risk reduction

NEN-EN-IEC 60204-1:2006

Safety of machinery – Electrical equipment of machines – Part 1 : General requirements

ISO/TR 14121-2

Safety of machinery – Risk assessment – Part 2: Practical guide and examples of methods

This installation also complies with the relevant parts of the RoHS Directive (2011 / EU) and the EMC Directive (13/40/ EU) by using CE marked components.

E.A. Meijer, Martini Beauty 01-09-2020



Extra options:

Article number	Description	Image
10007	Terry cloth seat cover PODO	
9000	Flexible Pedicure tray's	
10400	Lower Arm Support	
40200	Luxo magnifying lamp WAVE LED 3,5D	
30001	Buddy	
30011	Buddy with backrest Lende	
30021	Buddy with backrest Kiss	
30002	Ergoshape	
30002	Ergoshape Kiss	
30003	Jockey	
30103	Jockey with backrest Lende	
30004	ErgoShape 360 Round With back and arm support	
30005	ErgoShape 360 Plus With back and arm support	
30102	Barber-/ skater wheels	
60230	Feet support ring	

